Dermatix® Silicone gel for scar reduction

What is Dermatix gel?

Dermatix is a topical silicone gel that is transparent, dries quickly and helps maintain the skin's moisture balance, while aiding the management of scarred skin surfaces resulting from surgery, burns, and other injuries.

Dermatix has been shown to flatten, soften and smooth scars, relieve the itching and discomfort of scars as well as reduce the discoloration associated with scars.

Dermatix gel is easily applied to all areas of the skin, including the face, joints and flexures, and dries rapidly to form an invisible "sheet".

Dermatix was developed using the same basic long chain polymers as topical silicone gel sheeting, but it is available in a tube. It contains polysiloxane.

The gel is available in a tube of 15g.*

What is Dermatix gel used for?

Dermatix is used for the reduction and prevention of keloids and hyper-trophic scars (such as those resulting from general surgical procedures, trauma wounds and burns), after the wound has healed and the skin surface is intact.

How should Dermatix gel be used?

Wash the affected area and pat dry. Gently massage a small amount of Dermatix into the scar. If you apply too much gently wipe away any excess with a tissue in order to avoid staining of clothing. Apply twice daily in the morning and evening.

Once dry, Dermatix may be covered with cosmetic blush.

Dermatix can also be used on children.

When should Dermatix gel not be used?

- Dermatix should not be applied to open or fresh wounds.
- Dermatix should not be placed in contact with mucous membranes, or applied too close to the eyes.
- Dermatix should not be applied over antibiotic skin preparations or other skin treatments and/or products.
- Dermatix should be applied to clean, dry skin only.

Are there any possible unwanted effects of Dermatix gel?

Rarely, the gel may cause redness, pain or irritation to the skin. If you are concerned about these or any other unwanted effects, consult your pharmacist or doctor.

How should Dermatix be stored?

- Keep out of the reach of children.
- Keep gel away from the heat.
- Store below 25°C.
- Do not use after the expiry date (EXP) printed on the pack.

Further information.

You can get more information about Dermatix from your pharmacist or doctor.

Last revision date of the leaflet: 01/2020.

* Not all countries supply all presentations.

 **Hanson Medical, Inc., Kingston, WA 98346, USA**
Meda Pharma S.p.A., Via Valosa di Sopra, 9, 20900, Monza, Italy

EC REP

Distributor:

XIXIA PHARMACEUTICALS (Pty) Ltd.

4 Brewery Street, Isando, Johannesburg, South Africa, 1600

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