

## Reparil® Ice-Wrap

Cooling & Compression Bandage  
1 metre and 2 metre Bandage

### **FOR EXTERNAL USE ONLY**

For single use only:  
1 metre bandage for small injury areas and  
2 metre bandage for larger injury areas

### **What does Reparil® Ice-Wrap contain?**

Ethanol, Menthol, Arnica.

### **What is Reparil® Ice-Wrap used for?**

**Reparil® Ice-Wrap** is a cooling compression bandage for muscle and joint injuries.

### **Before using Reparil® Ice-Wrap, please take note of the following:**

For external use only.

Do not use if allergic to any of the ingredients.

Avoid contact with eyes, mouth, irritated skin and open wounds.

### **How to use Reparil® Ice-Wrap:**

- Tear open and remove bandage
- Squeeze lightly to remove excess fluid
- Wrap around injured area, overlapping the green line on the red line
- Tuck in or fasten to secure and leave uncovered to allow evaporation until dry
- Effective for over an hour. Replace when needed.

Store at or below 25° C.

### **KEEP OUT OF REACH OF CHILDREN**

### **DISTRIBUTOR**

### **XIXIA PHARMACEUTICALS (PTY) LTD**

4 Brewery Street, Isando, Kempton Park, 1600

Republic of South Africa

### **Date of revision:**

April 2014